




One System Basic Cloth Package

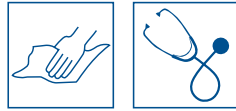

Single use wipe dispenser for filling with disinfectant. Contains 120 wipes.

Application

Generally in areas where the need for surface disinfection is frequent.

In the area of high hygiene standards: instrument maintenance, patient wards, operating theatres, polyclinics, dental care premises, in child health clinics and in other healthcare facilities.

For professional use.

Product properties

The hygienic, disposable package contains a roll of 120 nonwoven, low-linting wipes. When moistened with disinfectant, the dispenser can be used up to 28 days.

The One System Basic wipes combined with Easydes Disinfectant Surface Cleaner and Erisan Des Disinfectant have been tested for efficacy (EN 16615).

Wipe size 17.5 cm x 36 cm.

Instructions for use

Open the lid. Remove the protective inner covers. Pour 1,2 - 2.5 l of ready-to-use disinfectant solution over the wipes and allow to settle for 15 minutes.

Mark the data of the disinfectant you are using, date and filler identification on the package.

When using disinfectants, use protective gloves.

Country of manufacture

GERMANY

User and environmental safety

Serious incidents in relation to One system Basic wipes are to be reported to asiakaspalvelu@kiilto.com and the local medical device authority.

The package is recyclable, burnable plastic. Towels are of synthetic material (polyester).

Used wipes are hospital waste.

Package size / Product code

Sales unit	Product Code	GTIN	GTIN (sales unit)
4 x 120 cloths	8288	4260241964004	4260241966787

Additional information

Check the updated version of a paper brochure on our website at www.kiilto.com/en or send an e-mail to our customer service at asiakaspalvelu@kiilto.com.